

CARELIKA

PROFESSIONAL SKIN CARE



FACIAL ANTI-AGING SYSTEM MANUAL

For the best performance and safety, please read the manual before use. Keep the manual for future reference.

CONTENTS

SYSTEM MENU & ACCESSORIES	3
ACCESSORIES CONNECTION	4
SYSTEM ADJUSTMENT & DAILY MAINTENANCE	7
OPERATION INSTRUCTION	10
DERMABRASION OPERATION STEPS	10
SCRUBBER OPERATION STEPS	12
BIPOLAR RF OPERATION STEPS	13
MESOTHERAPY FACE OPERATION STEPS	15
MESOTHERAPY EYE OPERATION STEPS	16
COOLING TREATMENT OPERATION STEPS	17
BIO ROLLER OPERATION STEPS	18
BIO ROUND OPERATION STEPS	24
CONTRAINDICATION	26
SOLUTION FOR MALFUNCTION & PARAMETERS	27
PACKING LIST	28
IMPORTANT SAFETY NOTICE	29

SYSTEM MENU & ACCESSORIES


FRONT



BACK




ACCESSORIES CONNECTION

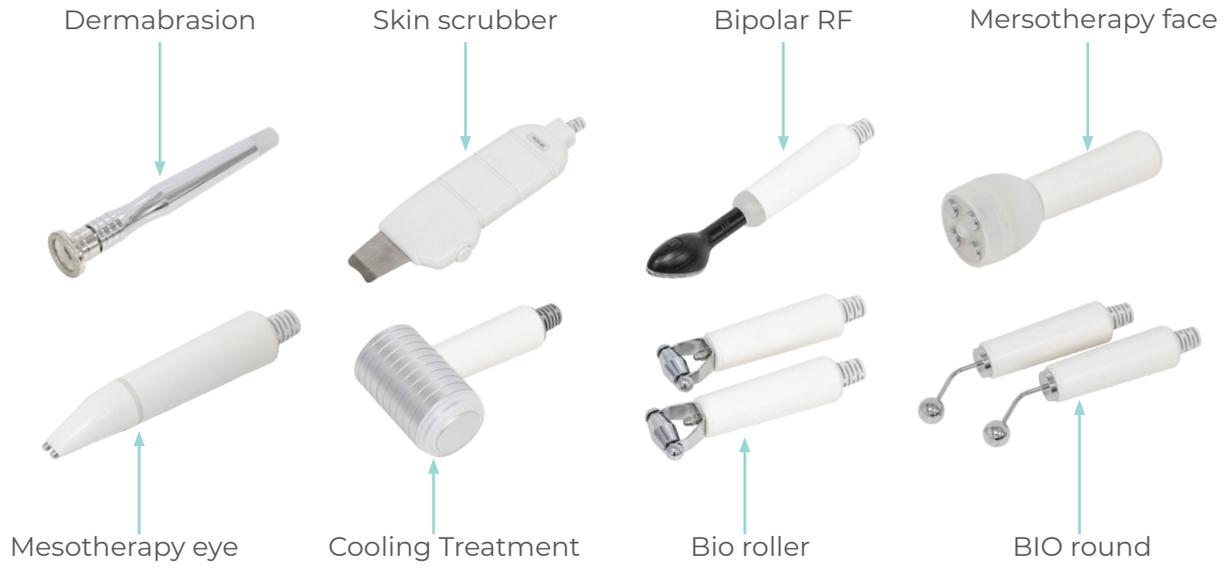


CONNECTION

- Dermabrasion connect with mainframe

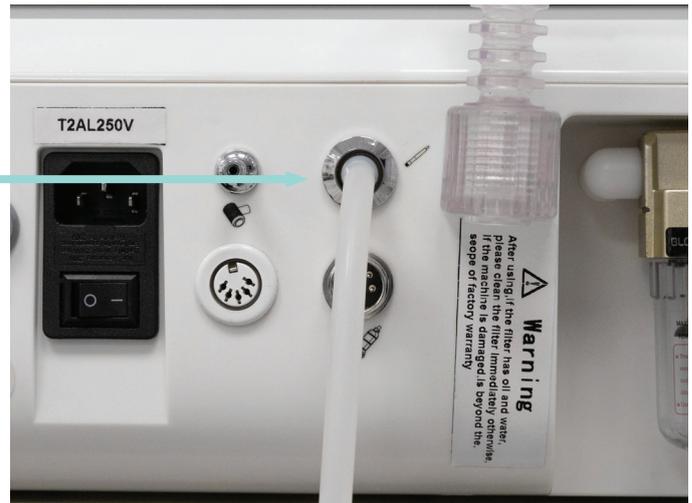

- Skin scrubber connect with mainframe

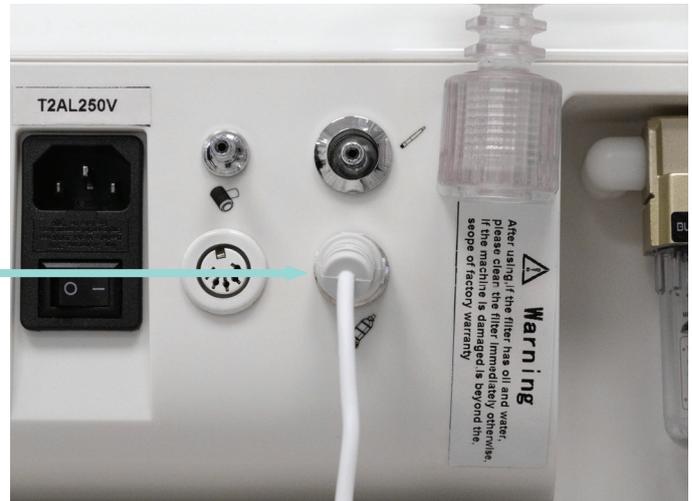


■ Bipolar RF connect with mainframe




■ Mesotherapy face connect with mainframe



■ Mesotherapy eye connect with mainframe

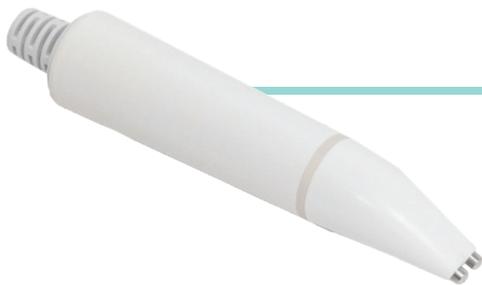

■ Cool hammer connect with the mainframe

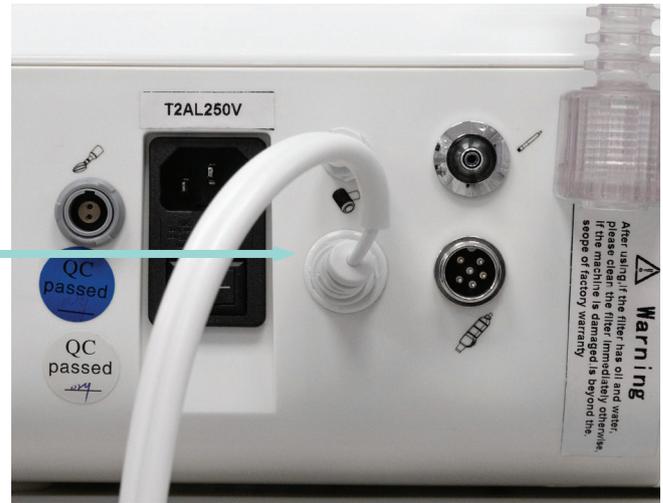

■ BIO roller connect with mainframe



■ BIO round connect with mainframe




SYSTEM ADJUSTMENT & DAILY MAINTENANCE

1. Turn on the main machine.
2. Connect main machine with the power line, turn on the switch, start the system, the screen show as below.




3. Press handle function to connect the chosen handle, the screen show as below.

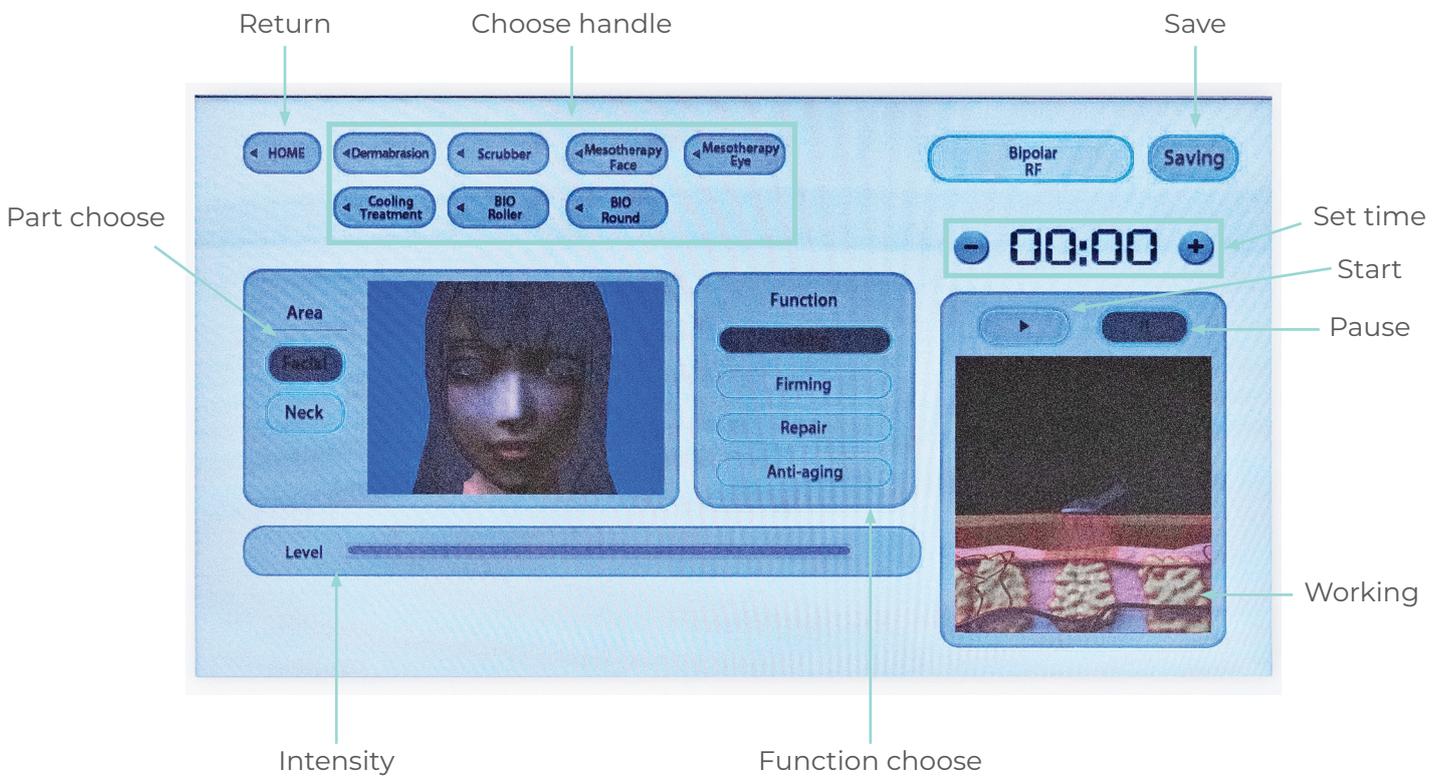


4. Choose part, choose function, set time mode and intensity.

5. After setting data press start and work.

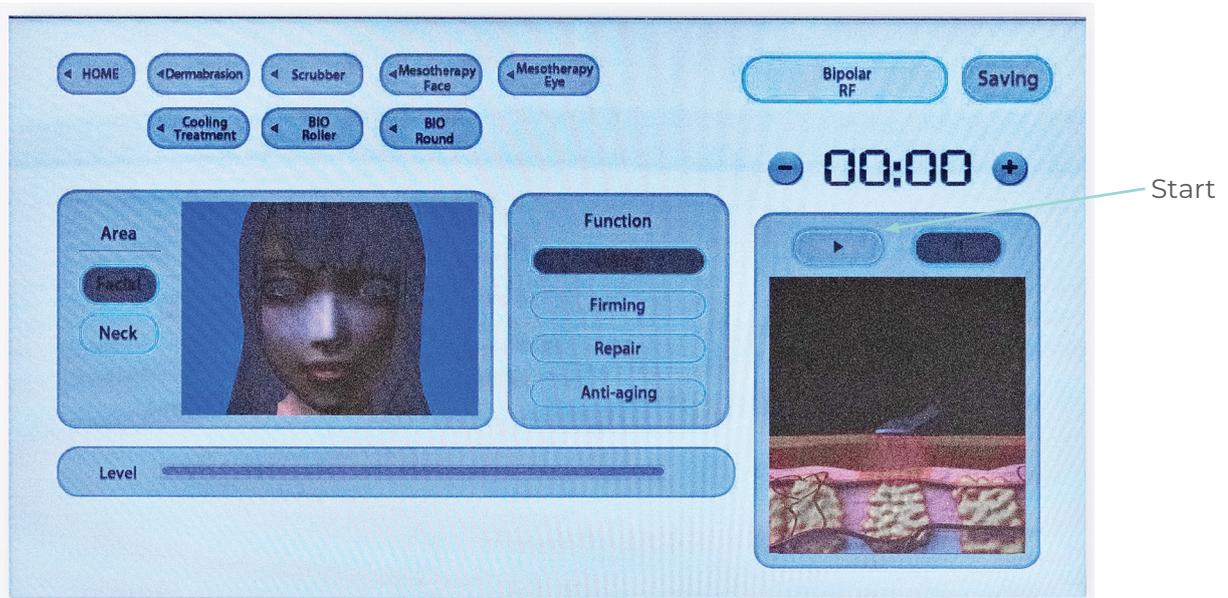

4. Press "SAVING", choose "YES", save the data

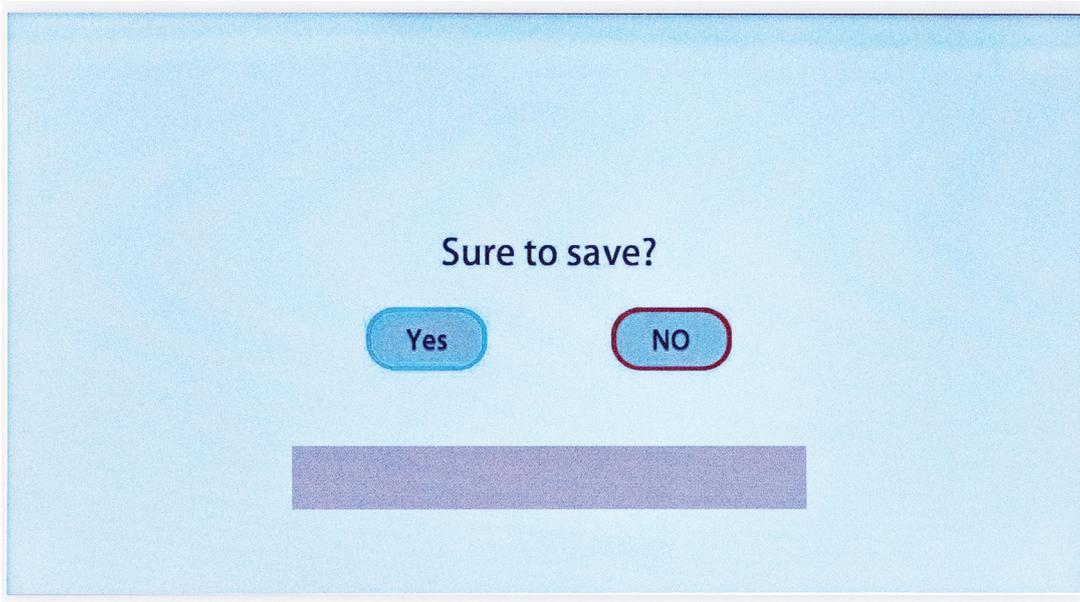

Warning:

In order to avoid unnecessary injury, please keep moving the working handles continuously on the skin surface. It is forbidden to hold the working handle on one place on the skin for a long time.

7. Scrubber Frequency Adjustment Step:

Turn on the power, show the main menu, click the upper right corner of the main menu, into the scrubber frequency adjustment page, adjust the frequency and save the data as the picture:

DAILY MAINTENANCE INSTRUCTION




Keep the machine and accessories clean. Wipe them clean with wet and soft cloth.



Disinfect the handles with 75% alcohol pad wipe before and after each operation.



Lightly touch the screen using the pulp of your finger to set it up.

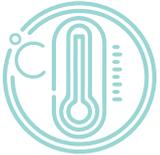

Working temperature: 5~55°C.



Environment humidity: not more than 80%.


OPERATION INSTRUCTION

DERMABRASION OPERATION STEPS




FACIAL

1. Choosing Dermabrasion spare part.
2. Adjusting work time 15-20 minutes; choosing the area and the function; adjusting the vacuum level which should be according to client's capability. As following:

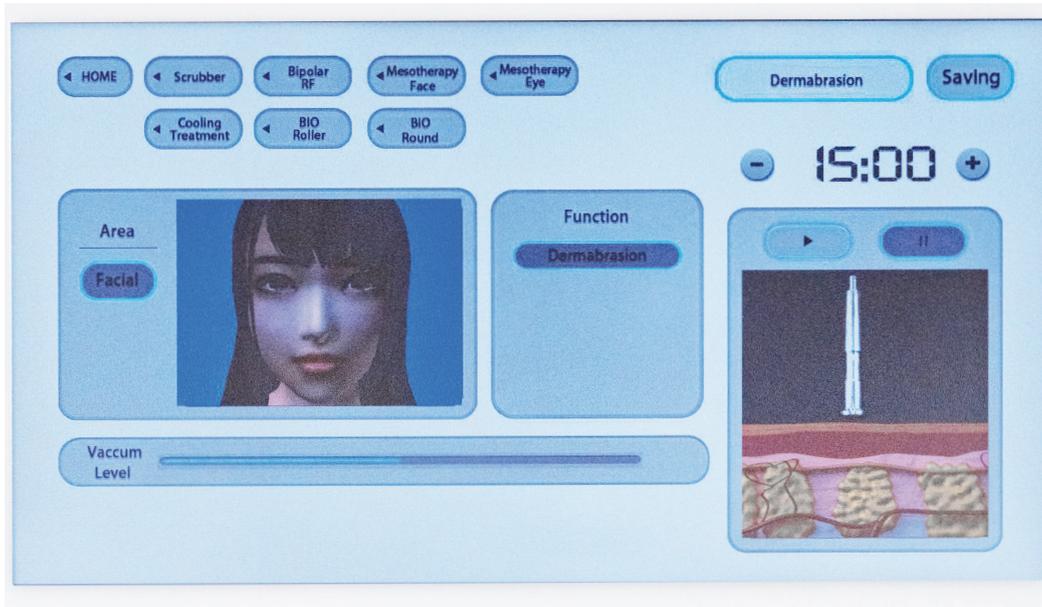

3. Facial. Move the probe as picture:

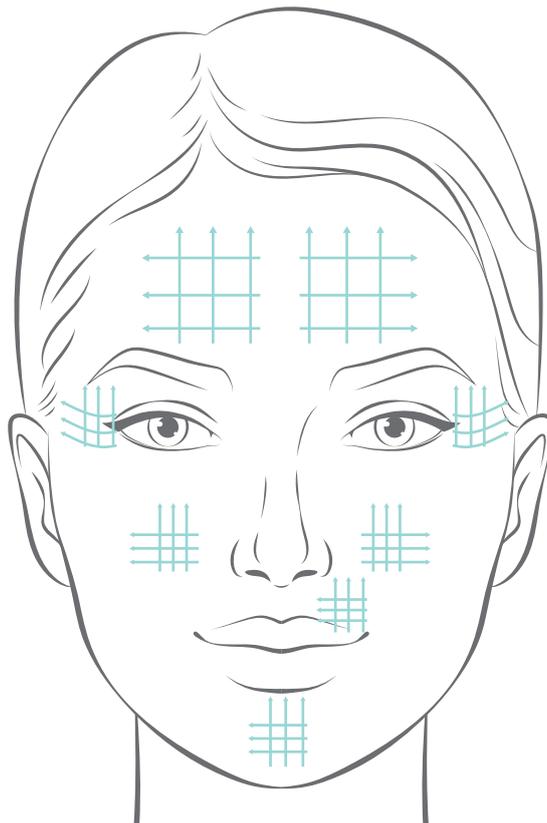

- If suction is too small, you have to re-replace filter during the operation. The number of different filters must be obtained by personal skin thickness of different areas and different sizes, each operation must be re-filled filters after the operation the peeling off skin can be taken to the customer.
- Please check whether the filters are in the right place, you can put some solid lubricant into the metal joint of the straw, such as vaseline, etc.
- Dermabrasion tips are of different size, Using tactile and visual sense to make sure what's a regional hypertrophy, generally, first choose the rough one to operate, and then the fine one.
- Suction adjusts. The thin the cortex, the lower the suction, you should set the lowest suction for the eye bag, crow's-feet etc.
- You should test the suction by arm or hand before operation. Get an average suction on the surface area. Be careful and don't leave the diamond tips over any area for more than 15 seconds. Otherwise, high pressure will lead to swelling or bleeding. Natural skin with direct line, and for the sensitive skin.
- After the operation, should use brush with water to clean diamond heads, after the cleaning, put it into the solution consisting of 75% alcohol for 15 minutes, then take it out and wipe it to complete dry. Put it into the ultraviolet radiation sterilizer for next using.


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

SCRUBBER OPERATION STEPS




FACIAL

1. Choosing Scrubber spare part.
2. Adjusting work time 15-20 minutes; choosing the area and choosing the function and the mode; adjusting the level which should be according to client's capability. As following:

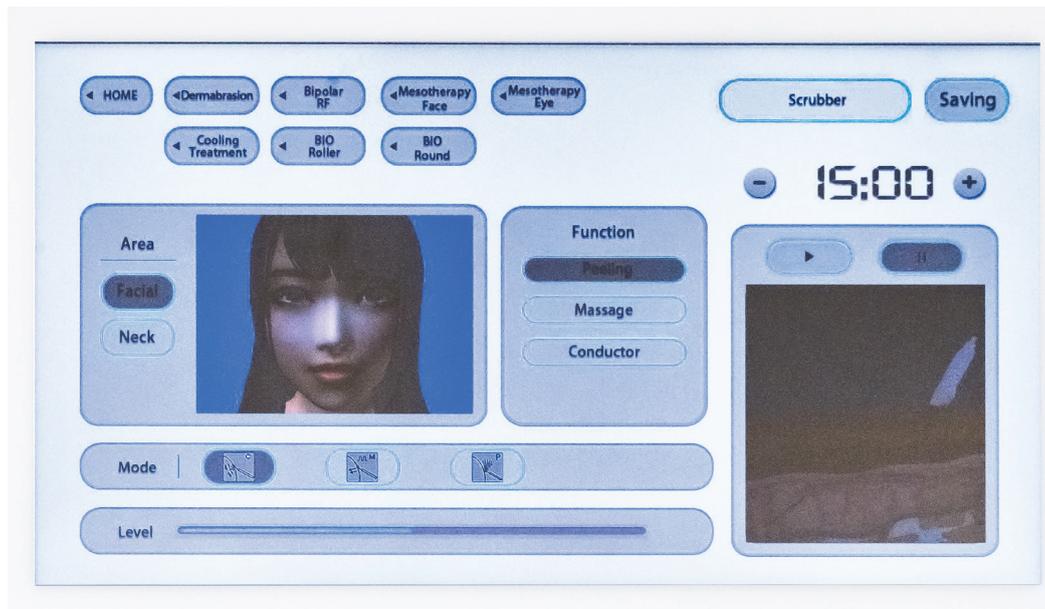


3. Supersonic skin scrubber is most popular with beauticians among several instruments. Trials for long period ensure its outstanding effect on facial care. It shifts electric vibration with 28000 times per second to mechanical vibration with 28000 times per second. Penetration effect of ultrasonic gives skin massage as well as cleaning. Meanwhile deep cells get excited. It can effectively remove spots, wrinkles, pimples, stratum corneum layer and residue in pores so that your skin looks white, purified and elastic.


Ultrasonic Massage

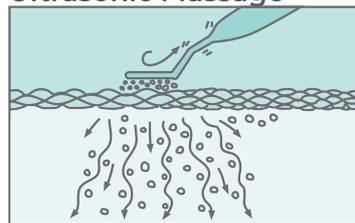


Remove wrinkles

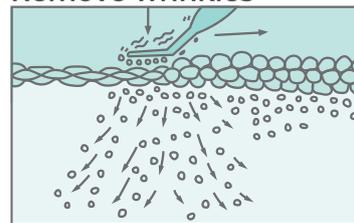

Peeling

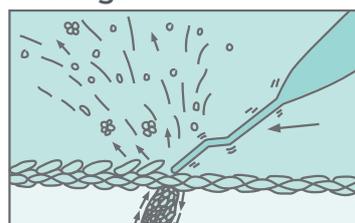

Remove melanin

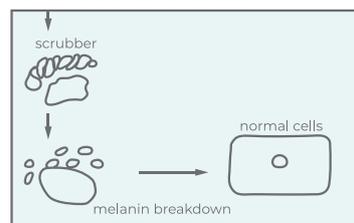


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

BIPOLAR RF OPERATION STEPS




FACIAL

1. Choosing Bipolar RF spare part.
2. Adjusting work time 15-20 minutes; choosing the area and the function; adjusting the level which should be according to client's capability. As following:

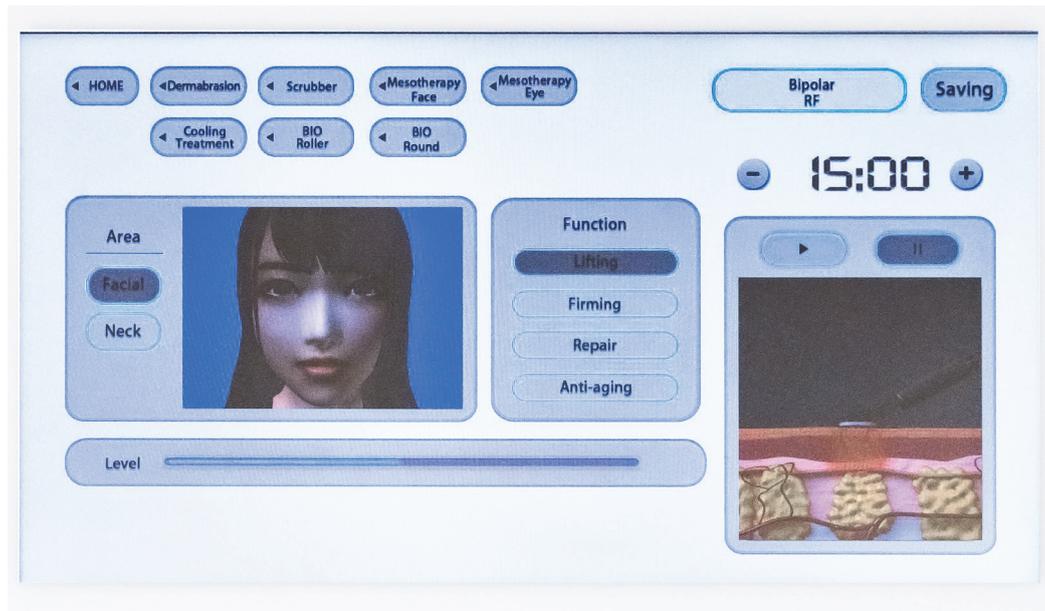


3. Before working, please put some medium on the area. Facial. Move the probe as picture:

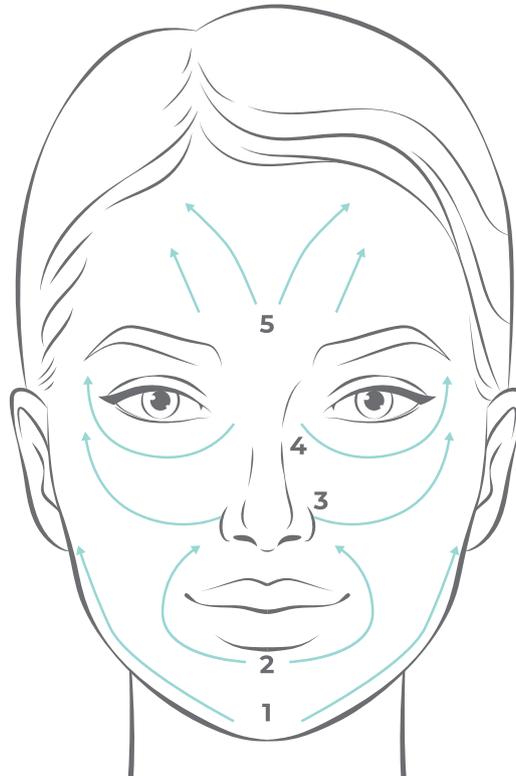


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling. The handle is forbidden to keep working on same place of the skin for a long time.

NECK

1. Adjusting work time 15-20 minutes; choosing the area and the function; adjusting the level which should be according to client's capability . As following

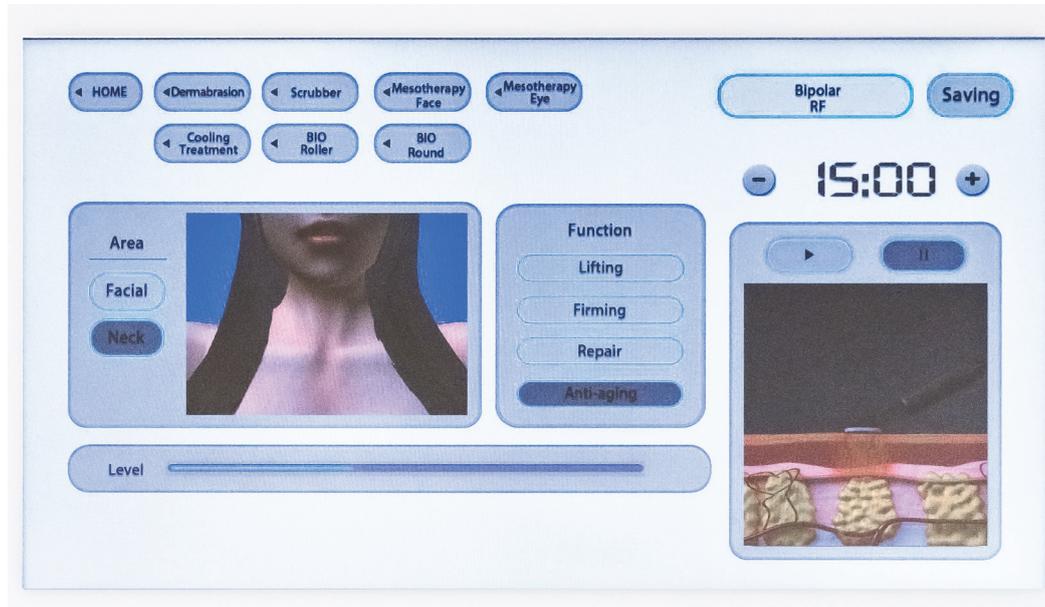

2. Before working, please put some medium on the area. Neck. Move the probe as picture:

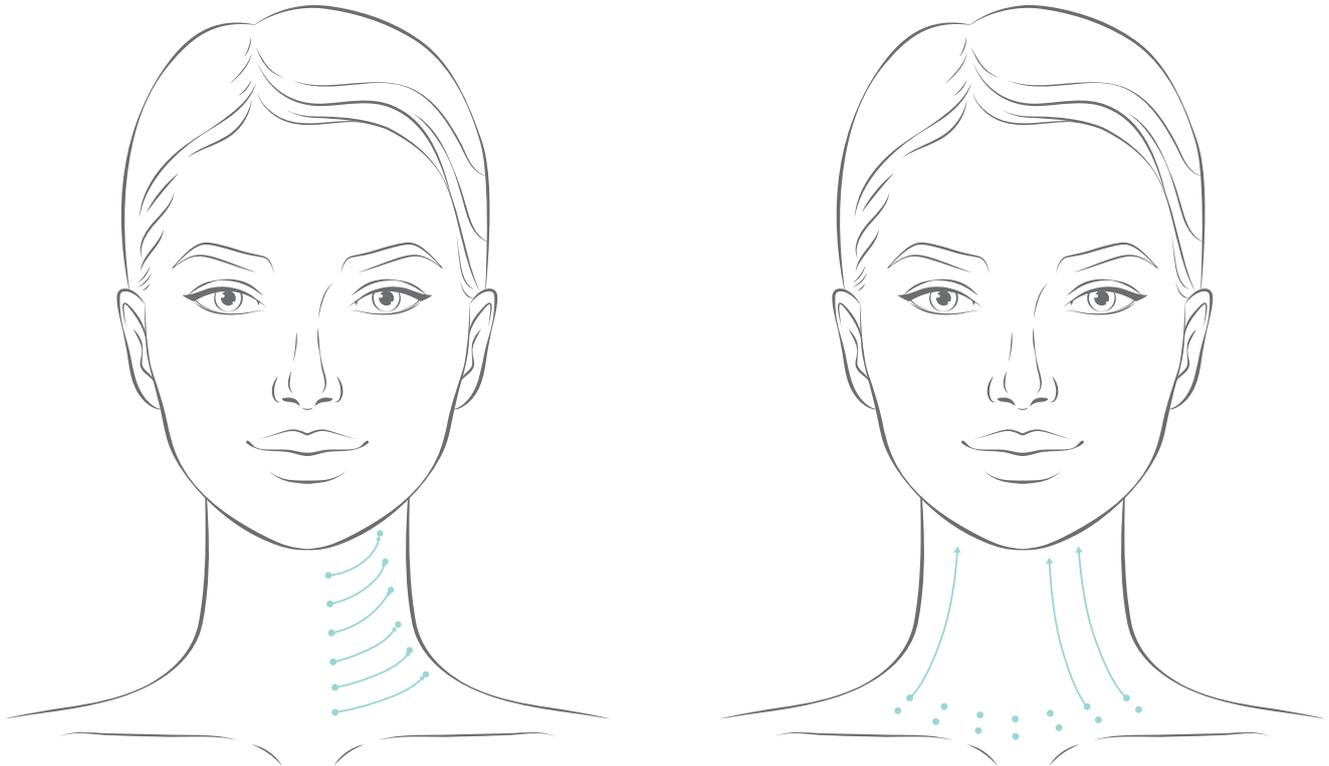


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling. The handle is forbidden to keep working on same place of the skin for a long time.

MESOTHERAPY FACE OPERATION STEPS



FACIAL

1. Choosing Mesotherapy Face spare part.
2. Adjusting work time 15-20 minutes; choosing the area and the function; adjusting the level which should be according to client's capability. As following:

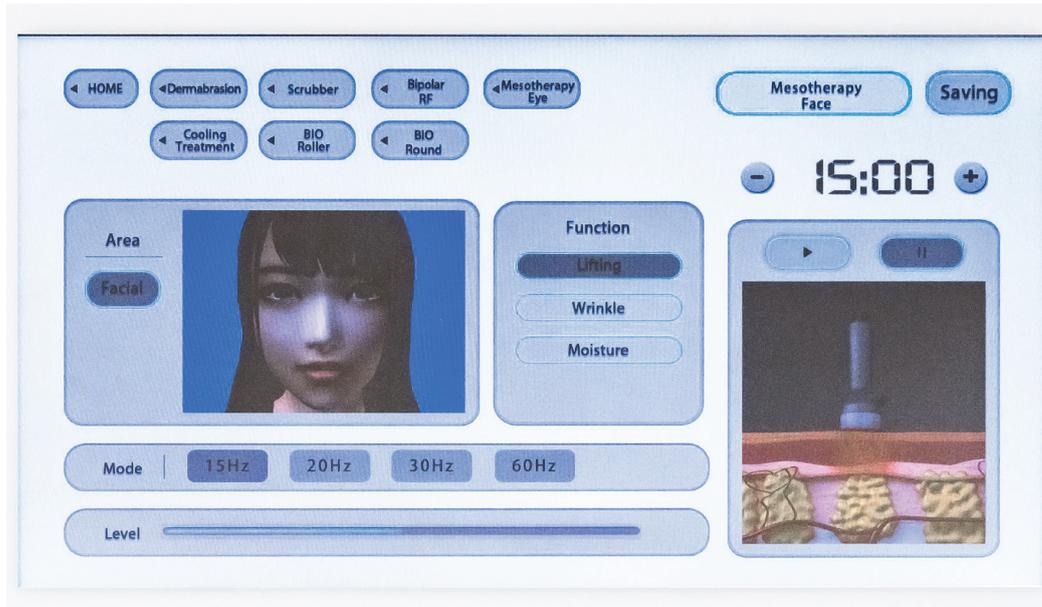


3. Before working, please put some skin care product on the area or put some skin care product into the handle's bottle. As the picture show:

(Twirling the handle, pull out the bottle, and put something into the bottle.)



4. Facial. Move the probe as picture.

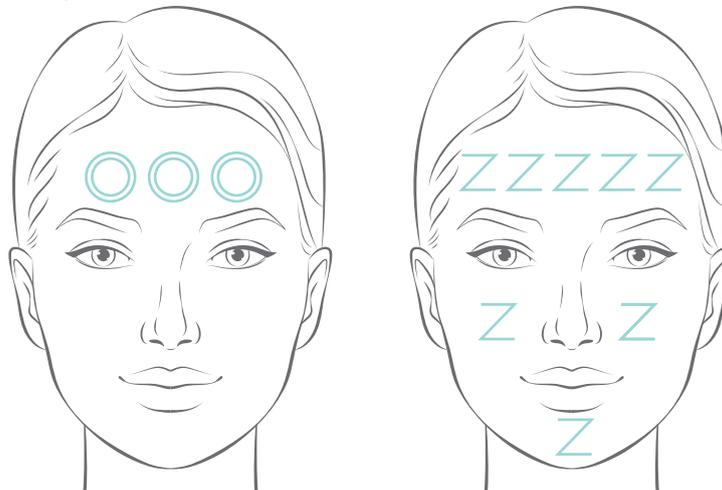


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

MESOTHERAPY EYE OPERATION STEPS




EYE

1. Choosing Mesotherapy Eye spare part.
2. Adjusting work time 10-15 minutes; choosing the area, choosing the function and the mode; adjusting the level which should be according to client's capability. As following:

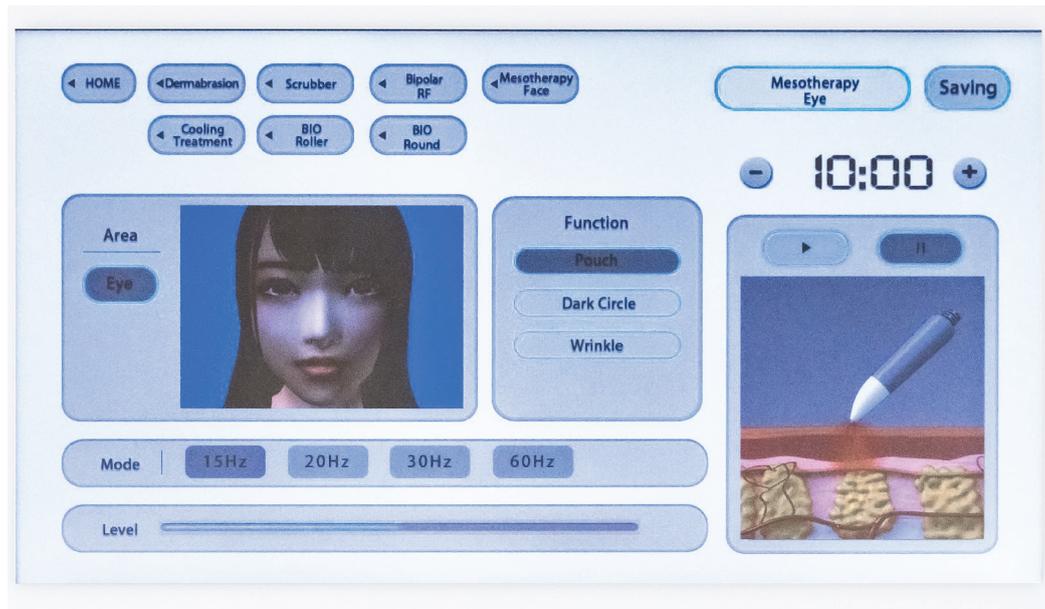

3. Eye. Move the probe as picture:

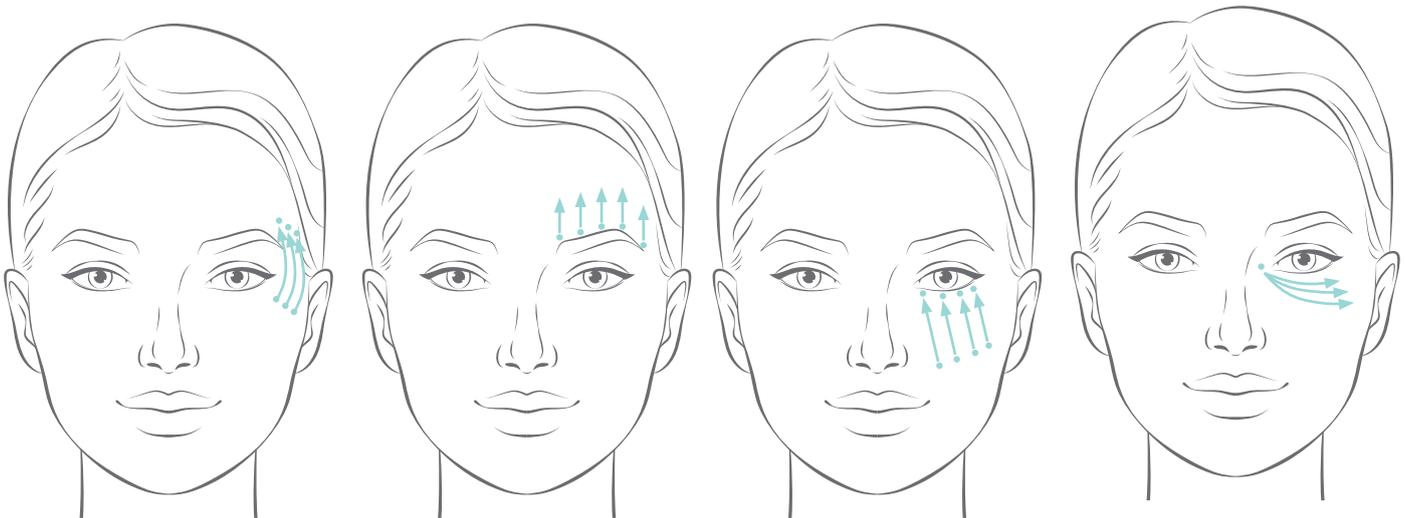


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

COOLING TREATMENT OPERATION STEPS




FACIAL

1. Choosing Cooling Treatment spare part.
2. Adjusting work time 10-15 minutes; choosing the area, choosing the function and the mode; adjusting the level which should be according to client's capability. As following:

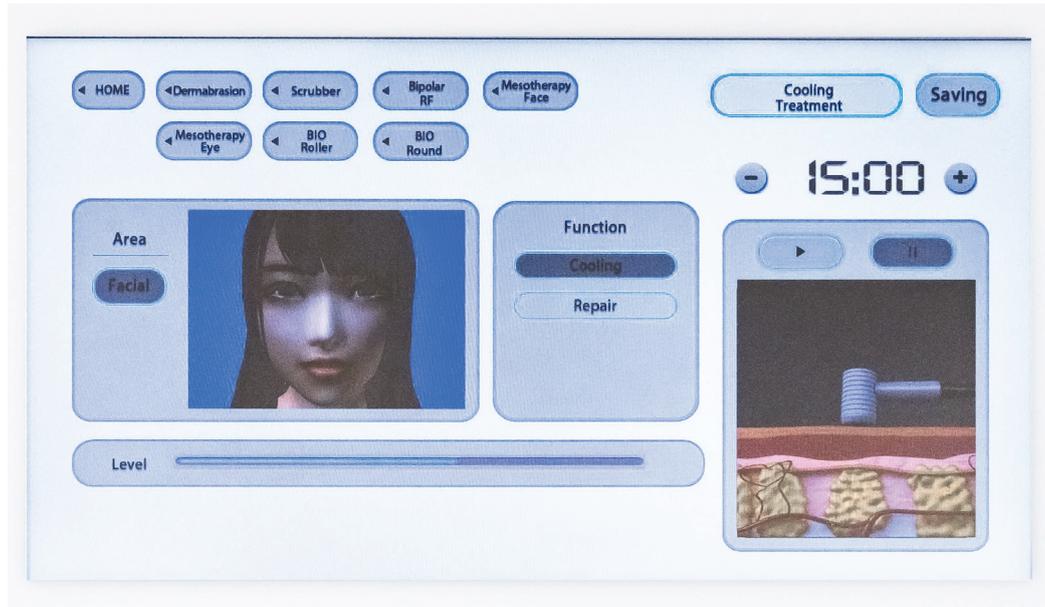


3. Facial. Move the probe as picture:

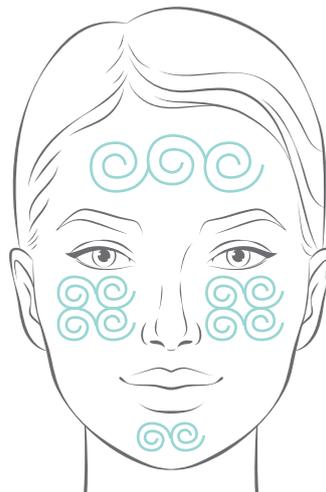


Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

BIO ROLLER OPERATION STEPS

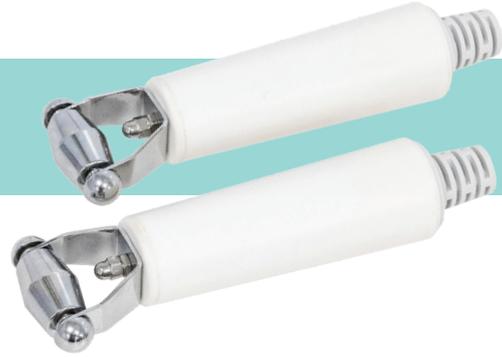

FACIAL

1. Choosing BIO Roller spare part.
2. Adjusting work time 10-15 minutes; choosing the area, choosing the function and the mode; adjusting the level which should be according to client's capability. As following:

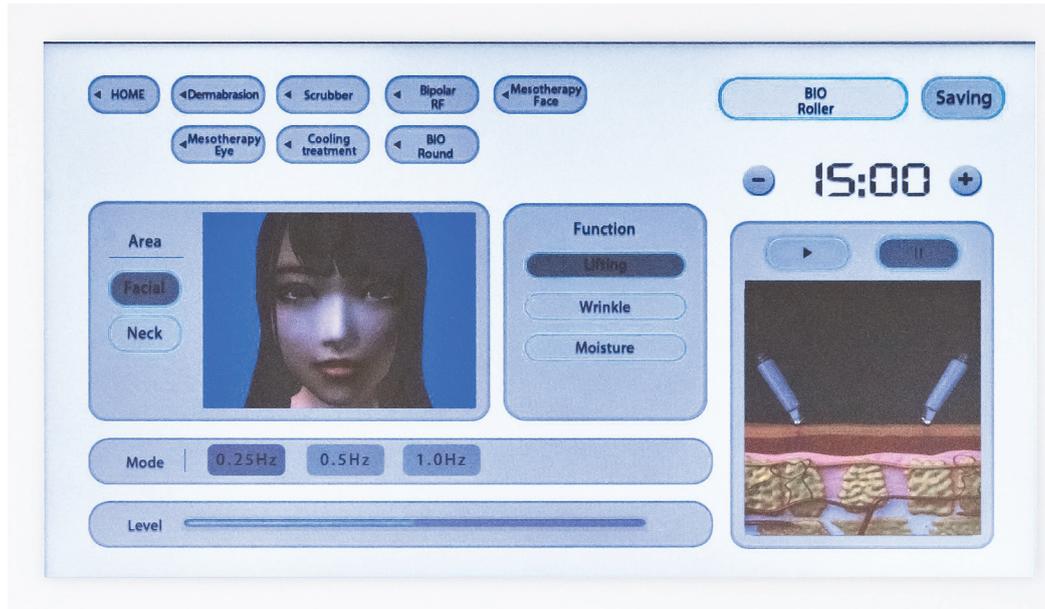


3. Operation Guide:


1. Exercises lour muscle to smooth away center wrinkling. Two probes draw at the same time from yintang to eyebrows; stop at Tsanchu then give press a little. The action repeats 3-5 times. (After an action, move a probe firstly till it touches skin then another probe follows; and hold acting continuity in the whole operation.)




2. Lengthening labium down muscle, musculi mentalis and mouth delta muscle in order to smooth away forehead wrinkling and chin wrinkling. This action draws with press bi-directionally from Chengchiang up and down separately and does 3 times beside left or right.

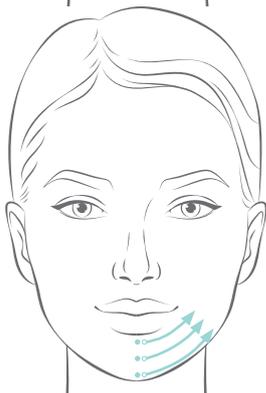


3. Lengthening surface skin to smooth vertical wrinkling. With the way of unidirectional draw with press, a probe locates a point where the probe shifts. From chin downwards for 3 times, another one slides from chin along with mouth corner onto Titsang, then pick with a little press up to cheekbone.

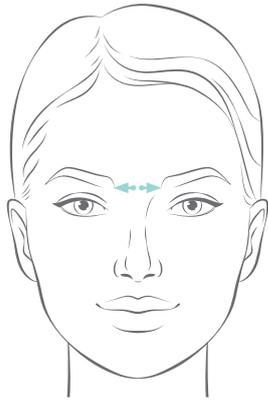


4. Exercise orbiculares round mouth to move way smile wrinkling. With the way of unidirectional draw with press, a probe sets onto Ch' engchiang as double march (which is step that after a motion and before the next motion, the probe is quickly picked up to the original point), another probe slide onto Chuliao for pause. The action repeats 3-5 times.

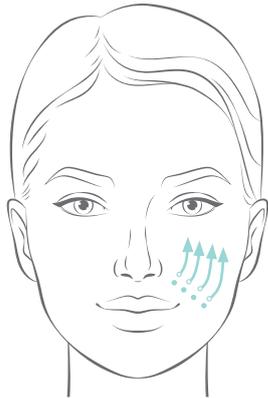


5. Shortening orbiculares round mouth. With the way of unidirectional extruding, a probe is set on the point of Jenchung (act as double march), another one slide from mouth corner onto nose wing behind. Two probes extrude each other. This action repeats 3-5 times. (Warning: do not touch the probes one to another)

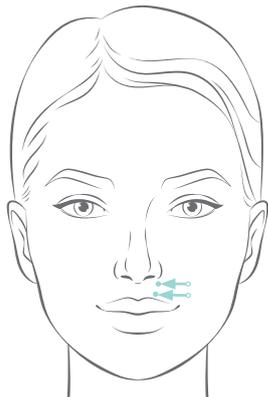


6. On labrum quadratus to smooth away laugh wrinkling. With the way of unidirectional extruding, a probe is set on point of Chingming (act as double mark), Another one slide from Tits'ang to Yinghsiang then pick up with a little force. This action repeats 3-5 times.

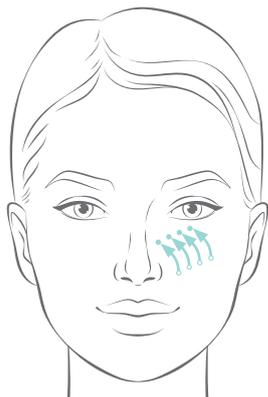


7. Exercise masseter to smooth away mandible wrinkling. With the way of bi-directional extruding, a probe sets to underside of cheekbone muscle; another one takes up masseter; move the probes from outside to inside for 4 times.

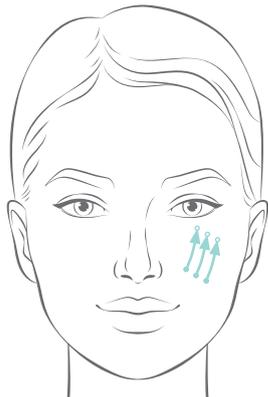


8. Exercise labrum quadratus to smooth away laugh wrinkling. With the way of unidirectional extruding, a probe set on inghsiang (act as double march) while another move along with Cingming, Ch'engch'i, Ch'iuhou then T'ungtzuliao for 4 minutes.

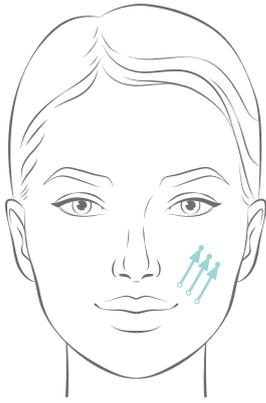


- 9.** Exercise masseter to smooth away mandible wrinkling. With the way of bi-directional extruding, a probe is set onto underside of cheekbone muscle and another one take up masseter; move the probes from outside to inside for 2 times.

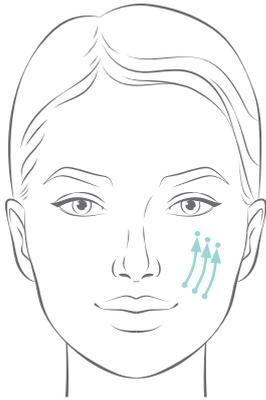


- 10.** Exercise big and smile cheekbone muscles to smooth away mandible wrinkling. With the way of bi-directional extruding, a probe take the acupoints of Ch'engch'i and Ch'iuhou, another one take up big and smile cheekbone muscle, move the probes from outside for 2 times.

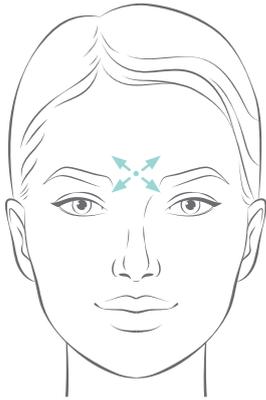


- 11.** Exercise cap-shaped aponeurosis and forehead muscle to smooth away paropia wrinkling. With the way of unidirectional extruding, a probe is set onto the place of one centimeter on top of brow end; another one slide up in camber from Ch'iuhou for 3 times (remarks: 3 times in camber, 3 times straight; while a probe takes up, another one take over the upward force but avoid touch each other)

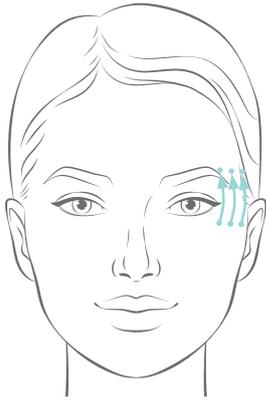


- 12.** Exercise lour muscle to smooth away center wrinkling, with the way of bi-directional draw with press, draw in horizontal for one time, then individually one time crosswise (from Yint' ang to Tsan-chu).

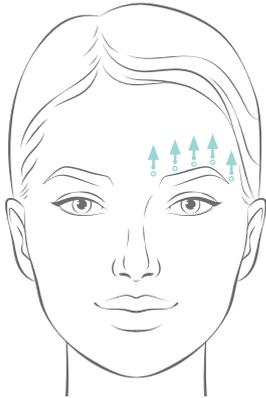


- 13.** Lengthening frons muscle to smooth away forehead wrinkling. With the way of bi-directional draw with press, two probes draw away up and down from middle of forehead, move to the left or the right for 5-6 minutes.

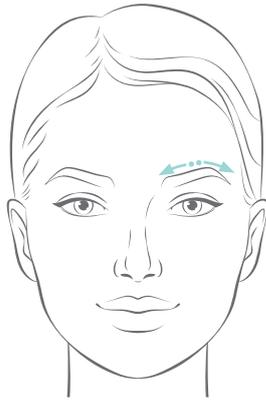


14. Lengthening Eye-socket orbiculares to smooth away center wrinkling. With the way of bi- directional draw with press, two probes draw meanwhile from Yuyao to Szuchuk'ung and Tsanchu then pick up a little. This action repeats 3-5 times.

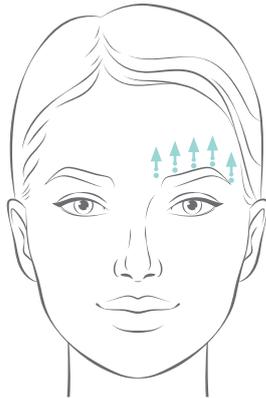


15. Raising frons muscle to smooth away forehead wrinkling. With the way of unidirectional extruding, a probe is set onto the place of one centimeter on top of brow end; another probe pick and draw to hair bound, move two probes to the left or right for 5-6 times.

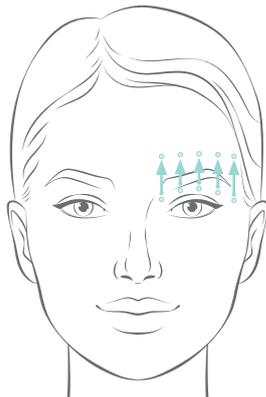

16. Raise brows, exercise brow drop muscle to smooth away paropia wrinkling. With the way of bi-directional extruding, a probe is set on top of brow, another one pick upward from underside of the brow; start from eyebrows to the end of brow for 5 minutes.

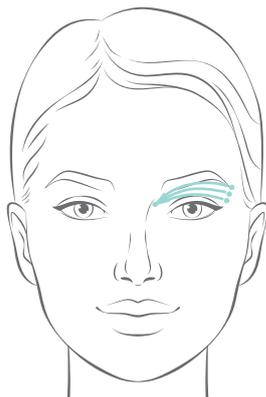

17. Tighten up eye socket orbiculares to smooth away pouch wrinkling. With the way of unidirectional draw with press, a probe is set onto T'ungtzuliao while another one slide from underside of eye in 45 degree from outside to inside of Chingming. This action repeats 3-5 times.

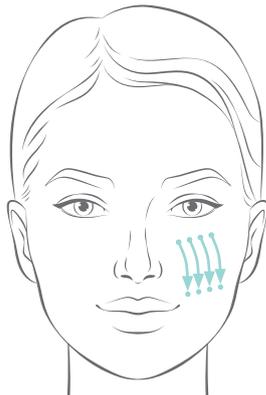


18. Tighten up eye socket orbiculares to smooth away pouch wrinkling. With the way of unidirectional draw with press, a probe is set onto T'ungtzuliao while another one slide from underside of eye in 45 degree from outside to inside of Chingming. This action repeats 3-5 times.

Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

NECK


1. Adjusting work time 10-15 minutes; choosing the area, choosing the function and the mode; adjusting the level which should be according to client's capability. As following:

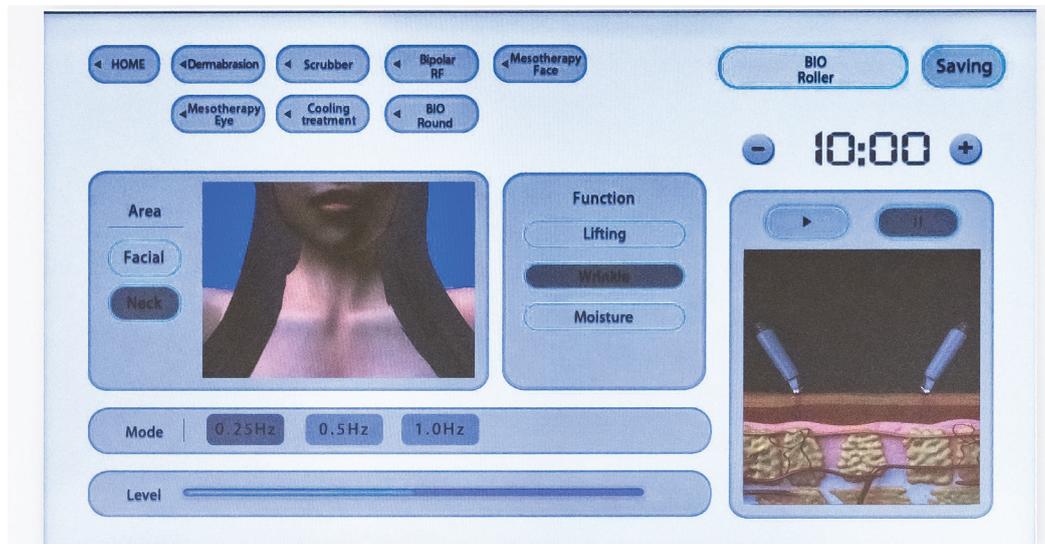


2. Anti-wrinkle operation on the neck:

Remark: wrinkle-off on the back must be after wrinkle-off on the face is completed for 6 times.


The first step: Normalizing exercise (Gentle wave in 6 minutes with 8 operations)

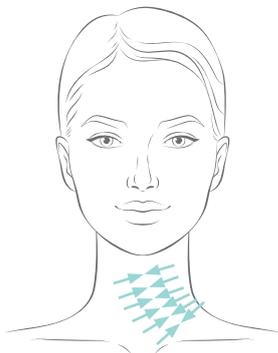


1. Exercise left sternocleidomastoid to smooth wrinkle on the neck. By the way of bi-directional extruding, moving from the top to the bottom of the neck;

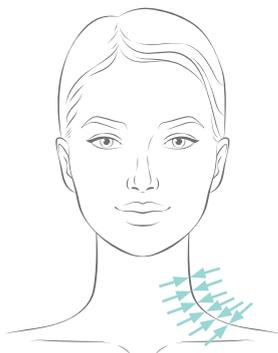

2. Repeat operation 1 but towards the right side;

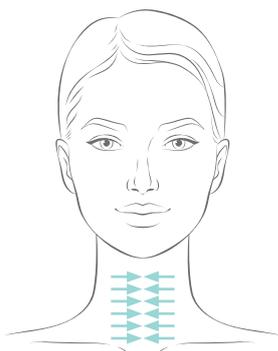


3. Exercise muscili platysma to remove double chins. By the way of bi-directional extruding, moving from bottom to top of the neck;

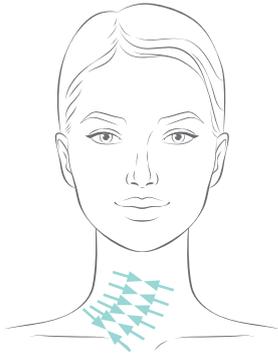


4. Exercise the right sternodeidomastoid to remove wrinkles on the neck. By the way of Bi-directional extruding, moving from the top to the bottom of the neck;


5. Repeat operation 4 but towards the right side;

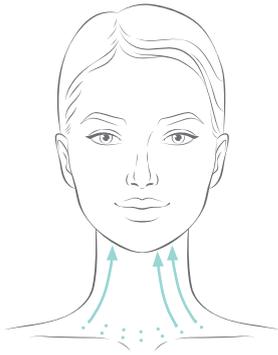

6. Lengthen muscles on the neck to remove its wrinkles. By the way of Bi-directional press and pull, two probes draw away at the same time up and down from the middle of the neck, move from the left to the right for 10 times;

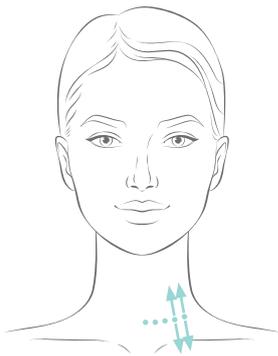

7. Hold up the left sternoleidomastoid and musculi platysma to remove wrinkles on the neck. By the way of unidirectional extruding, one probe is set onto the top of the neck while another extrudes up wards with moving from the left to the right side for 10 times;

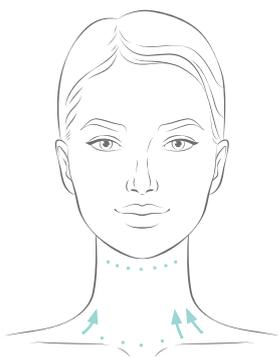

8. Exercise neck muscles to remove the neck wrinkle. By the way of unidirectional press and pull, one probe is set without move while another one slides upwards from the left to the right side for 10 times.

The second step: exercise to increase spring of neck muscles (pulse wave for 8 minutes, with 3 motions)

Remark: same as the operation 6, 7, and 8 of the first step.

The third step: Deep wave. (mild wave for 3 minutes, with 3 motions).

Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

BIO ROUND OPERATION STEPS




EYES


1. Choosing BIO Round spare part.
2. Adjusting work time 10-15 minutes; choosing the area, choosing the function and the mode; adjusting the level which should be according to client's capability. As following:




3. Operation Guide:

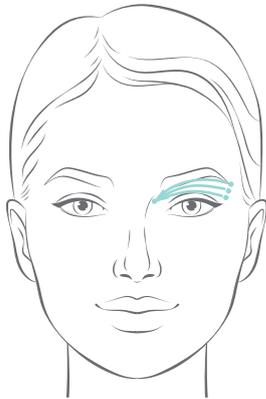


1. Tighten up eyes socket orbiculares to smooth away pouch wrinkling . With the way of unidirectional draw with press , a probe is set onto T'ungtzuliao while another one slide form underside of eye in 45 degree from outside to inside of Chingming . This action repeats 3-5 times.

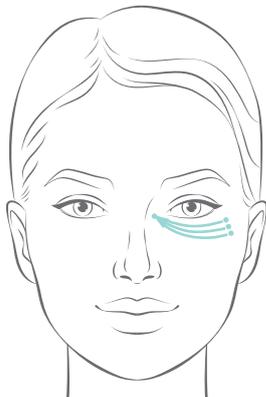


2. Tighten up eye socket orbiculares to smooth away pouch wrinkling . With the way of unidirectional draw with press ,a probe is set onto T'ungtzuliao while another one slide from topside of eye in 45 degree from outside to inside of Chingming . This action repeats 3-5 times .

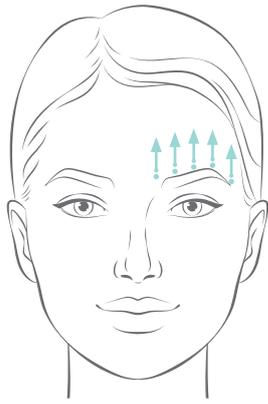

3. Exercise masseter to enhance its elasticity . With the way of bi-directional extruding , a probe is set onto the underside of cheek-bone muscle, another one raise masseter from outside to inside for 2 times .

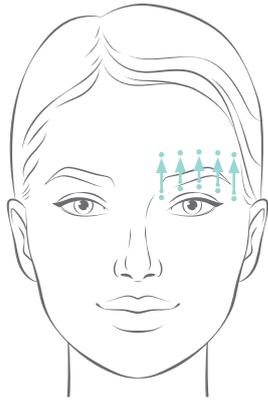

4. Lengthening Eye-socket orbiculares to smooth away center wrinkling. With the way of bi-directional draw with press, two probes draw meanwhile from Yuyao to Szuchuk'ung and Tsanchu then pick up a little. This action repeats 3-5 times.

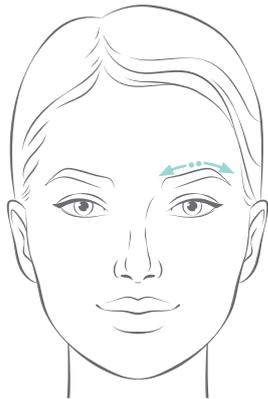

5. Raise brows, exercise brow drop muscle to smooth away paropia wrinkling. With the way of bi-directional extruding, a probe is set on top of brow, another one pick upward from underside of the brow; start from eyebrows to the end of brow for 5 minutes.

Note: The above data is for reference only, please communicate with client during treatment and adjust the data according to client's actual feeling.

CONTRAINDICATION

Do not use in the following cases:

1. The people with pacemaker or internal defibrillator;
2. The operation body part has metal implant (such as gold wire, metal teeth);
3. The people with skin cancer or precancerous lesions;
4. The people with severe hypertension, diabetes, heart disease, etc;
5. Pregnant or lactating women;
6. The operation body part with bacteria, viruses, open wounds, etc;
7. The operation body part with permanent fillers, or temporary fillers (consult your doctor first);
8. Recently (in 3 months) treatment with hydroxy acid, laser skin resurface, chemical exfoliation or wound healing;
9. Do not use blood anticoagulant or isotretinic acid (retinoic acid) taken in the past 2 months;
10. It is forbidden to hold the working handle on one place on the skin for a long time.

SOLUTION FOR MALFUNCTION & PARAMETERS

Failure	Solution
Power on, the machine is not responding.	Check whether the power supply is in good connection with main machine. Check whether the power switch is on.
No output.	Check whether the accessory is in good and right connection with the output port of main machine.
During working, the main machine is overheating.	If used for too long, turn off and cool down naturally. Check whether electricity voltage is correct (need professional workers to check).
No power.	Check the fuse, if broken it is needed to replace it.

Note: To avoid interference, please don't operate two or more than two functions at the same time.

PARAMETERS

Packing size	77*45*35 cm
Power	75W
Voltage	220V 50Hz
Electric current	2A
Net weight	12 kg
Gross weight	15 kg

PACKING LIST

Main machine	1 pc
Mesotherapy face	1 pc
Mesotherapy eye	1 pc
BIO Round	1 pc
BIO Roller	1 pc
Bipolar RF	1 pc
Cool hammer	1 pc
Dermabrasion	1 pc
Skin scrubber	1 pc
Dermabrasion box	1 pc
Pipe	1 pc
Shelf	1 pc
Support Bar	2 pcs
Screw	2 pcs
Spanner	1 pc
Wire	1 pc
Fuse	2 pcs
Manual	1 pc

IMPORTANT SAFETY NOTICE

1. This machine should be operated by professional beautician only.
2. Do not disassemble the machine or operate the machine that not explained in the instruction book. All the maintenance services should be carried out by professional maintenance technician.
3. Do not install or operate the machine with wet hands or near the water source so as to avoid the liquid spraying/swashing over the machine.
4. Pull out the plug and connect with the maintenance services center with following conditions:
 - 4.1. Liquid spray/swash over the machine or the machine get caught in rain or water;
 - 4.2. The machine has peculiar smell, fume/smog or abnormal noise;
 - 4.3. Instrument fell down or the shell damaged;
 - 4.4. Power wire damaged or scratched.
5. Do not put any item on the power wires, or install the instrument in the place where its power wires will be trampled.
6. Pull out the power wires at the end of use for the sake of safety.
7. Do not insert any type of item into the instrument opening, to avoid a fire or electrical shock. In case if something fell into the opening but you could not take it out, please pull out the power wires and connect with the dealer or our Company.
8. Do not put the instrument on the unstable trolley, shelf or vehicle, as a fall may damage the instrument.
9. To guarantee the life circle of this machine, the interval of the continuous switch on and off should be over one minute.
10. The power shall comply with the specification specified in the label, otherwise the machine will not work or work abnormally, even be burnt out.
11. The company has right to revise or amend this manual, and reserves all the rights of interpreting the information mentioned above.
12. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved;
13. Children shall not play with the appliance;
14. Cleaning and user maintenance shall not be done by children without supervision.

CARELIKA

PROFESSIONAL SKIN CARE

 carelikaprofessional
 Carelika Professional

www.carelikacosmetics.com